


BLUETOOTH REMOTE CONTROL MANUAL

Open smart life with one click

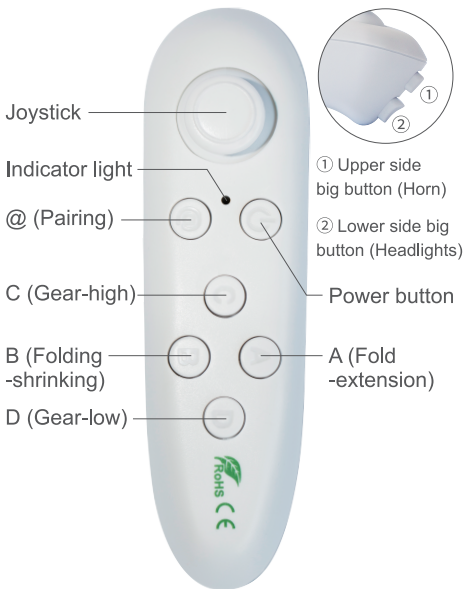

Key Description

INSTRUCTIONS

How to use

1. **Install battery**
Prepare two No. 7 dry batteries, open the remote control rear cover, and install it correctly;
2. **Connection wheelchair**
Short press the power button (such as the power button), the indicator light flashes green, start to connect the wheelchair, about 3-5 seconds, wait for the wheelchair to send a "click", indicating that the connection is successful, can be used to control the wheelchair;
3. **Rocker control wheelchair**
Move the rocker (see picture) to control the wheelchair forward, backward and turn;
4. **Speaker control**
Short press or long press the button on the front side (such as the upper side big button - speaker), control the speaker short or long
5. **Headlight control**
Short press the front side down button (such as the big button on the lower side - headlights) to turn the headlights on or off;
6. **High and low gear adjustment**
Short press C key (as shown in Figure C-gear-high) to adjust to high-end position, short press D key (as shown in Figure D-gear-low) to adjust to low gear;
7. **Folding telescopic wheelchair**
 - a. Short press the power button (such as the power button) to shut down;
 - b. Press and hold the A button (as shown in Figure A-automatic expansion) or the B button (as shown in Figure B-automatic folding) for about 3 seconds, enter the folding telescopic mode, short press the A or B button to stop or continue to expand and contract;
 - c. Short press the power button (such as the power button) to exit the folding telescopic mode.

Indicator light

1. During use, the indicator light flashes green slowly, indicating that everything is normal;
2. During the connection process, the indicator light is red, indicating that the connection is wrong. Please try to connect the wheelchair several times.
3. During normal use, the indicator light is red, indicating that the battery is low. Please replace the new No. 7 dry battery.

Remarks

The Bluetooth remote control has been paired when it is shipped from the factory. If it cannot be connected properly, please follow the instructions below to re-pair.

Pairing instructions: Move the remote control within 1 meter of the wheelchair controller, press the "Power button" briefly, the green light is always on, then press the "@" button quickly, the green light starts to breathe slowly and enter the pairing (do not move the joystick during the pairing process).); After the pairing is successful, the green light flashes quickly and enters the remote control mode.